

Programming Advantage 1000

CONTENT IS SUBJECT TO CHANGE WITHOUT NOTICE



SECTION 8 PROGRAMMING INSTRUCTIONS

NOTE: All Panels shipped from Secure Care Products®, Inc. after March 5, 2004 have a non-volatile memory, which means any programming of the panels will not be lost after power loss. To initialize the panel enter *603 *8675309999. Nine beeps should be heard and all programming returned to factory settings.

User Selectable Functions

Factory Default

Primary Reset Code: any 3 digits preceded by *

***234**

To change the primary reset code, enter *567 (secondary code) *9876543210#xxx#, one beep should be heard. (The xxx is the new three-digit number.)

Secondary Code: any 3 digits preceded by *

***567**

To change the secondary reset code, enter *567 (secondary code) *9876543211#xxx#, one beep should be heard. (The xxx is the new three-digit number.)

Authorized Entry Time: 15, 30, 60, or 120 seconds

30 Seconds

To change, enter *567 (secondary code) * then the corresponding number (hold for one beep)

1= 15 seconds 2= 30 seconds 3= 60 seconds 4= 120 seconds

PM Program Enable/Disable:

Disabled

To enable PM Program enter *567 (secondary code) *7 hold until four beeps are heard.

To disable PM Program enter *567 (secondary code) *8 hold until one beep is heard.

Loiter Alarm Enable/Disable:

Disabled

To enable enter *567 (secondary code) *5 hold until four beeps are heard.

To disable enter *567 (secondary code) *5 hold until one beep is heard.

Loiter Time:

60 seconds

To change to 60 seconds enter *567 *60, one beep will be heard.

To change to 120 seconds enter *567* 12, two beeps will be heard.

To change to 180 seconds enter *567*18, three beeps will be heard.

Latching Delayed Egress: (Enabled or Disabled)

Disabled

To enable enter *567 *0 holding until 4 beeps are heard.

To disable enter *567 *0 holding until one beep is heard.

Advanced Security Mode: (Enabled or disabled)

Enabled

To enable Advanced Security Mode: enter *567 (secondary code) *9876543216 holding the 6 until two beeps are heard

To disable Advanced Security Mode: enter *567 (Secondary code) *9876543216 holding the 6 until one beep is heard.

Programming the PM Feature

Enter *567 (secondary code) then enter *987654321 holding the 1 until one beep is heard

Enter the current time of day in military time

Example 6:00 AM = 0600, two beeps

Enter the PM off time

Example 5:30 AM = 0530, three beeps

Enter the PM on time

Example 9:00 PM = 2100, four beeps

The PM programming is now entered into the Advantage 1000 panel, but must be enabled before the PM feature will work. Follow the PM Program enable instructions above to enable the PM Program.

Software Verification: To verify the version of software installed in the panel, enter *567 (secondary code) *9876543215 count the number of times the LED's blink. This will be the software version.

Example: 3 blinks...pause...4 blinks...pause...10 blinks...pause...9 blinks, would be Software Version 3.4.0.9